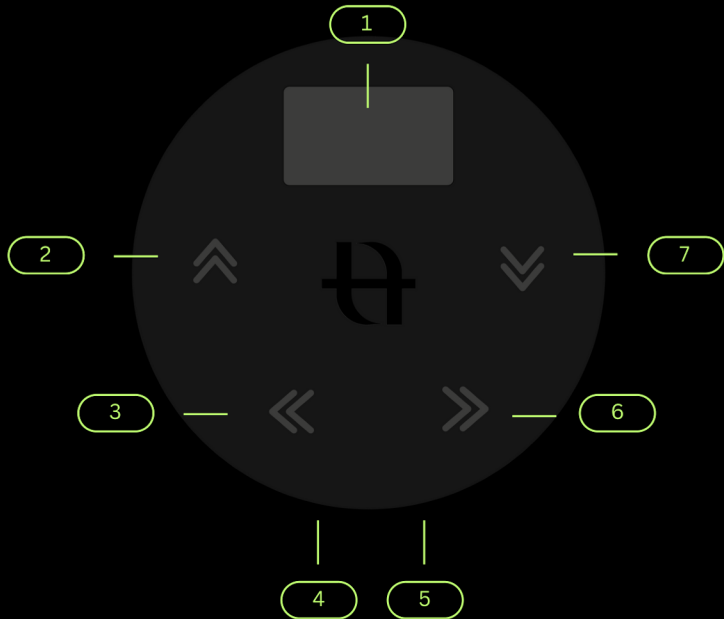
The background of the image is a black, wavy, textured surface that resembles draped fabric or a liquid surface. The waves are dark and create a sense of depth and movement. In the center, the word "TITAN" is written in a large, bold, white, sans-serif font. To the right of the "N" is a small white copyright symbol (©). Below "TITAN" is the phrase "power supply" in a smaller, lowercase, green, sans-serif font.

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LEGEND

1

The TITAN PMU power supply features a built-in LED display.

2+7

The Titan power supply has a built-in flashing backlight to indicate the selected voltage during use.

3+6

When adjusting the voltage using the up or down buttons on the power supply, the increment is 0.1V, ranging from 3.3V to 15V. The output current can be set at 1A or 2A, and the maximum output current varies with different voltage settings.

4+5

The power supply features a built-in dual-machine running mode. Through the setup options, the foot pedal input can be configured to serve as a secondary machine input.

USE

To use our power supply for a PMU machine, follow these steps:

1. *Connect the power supply to an outlet.*
2. *Connect the power supply to the PMU machine.*
3. *Set the voltage of the power supply to the desired value.*

The voltage of the power supply determines the speed and power of the PMU machine. A higher voltage results in a faster and more powerful machine. If you are using a PMU machine with an adjustable frequency, you can also set it with the power supply. The frequency determines the depth of penetration of the needle. A higher frequency results in a greater depth of penetration.

Here are some specific tips for using our power supply for a PMU machine:

- *Always start with a low voltage and gradually increase it as you gain experience.*
- *Always use a needle that is suitable for the desired voltage.*
- *Follow the safety instructions for the power supply and the PMU machine.*

Here is a more detailed explanation of the steps:

- **Step 1: Connect the power supply to an outlet/** The power supply has a plug that you can plug into an outlet. Make sure that the outlet has a voltage of 220-240 volts.
- **Step 2: Connect the power supply to the PMU machine /** The power supply has an RCA connector that you can plug into the RCA port of the PMU machine. Make sure that the RCA connectors are compatible.
- **Step 3: Set the voltage of the power supply to the desired value /** Most power supplies have a knob or slider that you can use to set the voltage. The desired voltage depends on the type of PMU machine, the needle you are using, and the technique you are using. If you are not sure which voltage to use, consult the manual of the PMU machine.

Here are some general guidelines for the voltage of a PMU machine: After you have set the voltage, you can start applying permanent makeup.

- For eyebrows: 5-8 V
- For eyeliner: 8-10 V
- For lips: 10-12 V

How to maintain your power supply

The power supply should be cleaned and disinfected regularly. Make sure the device is turned off before cleaning. Use a mild detergent and a soft cloth.

Safety

The power supply is equipped with safety features to prevent overload.

Complaints

If you have a complaint about your power supply, please contact the supplier.

- Info@titanpmu.com
- titanpmu.com
- IG @pmu.titan

WARRANTY

Your Titan power supply comes with a 1-year warranty. This means that if a defect occurs within 1 year, we will repair or replace the device free of charge.

Conditions

The warranty applies only if the device is used and maintained correctly. This means that you must use the device according to the instructions in the manual. You must also clean and disinfect the device regularly.

The warranty does not apply to damage caused by:

- *Use with a PMU device other than the Titan PMU device.*
- *Dropping the device.*
- *Other forms of abuse or neglect.*

Repair costs

The warranty expires after 1 year. If damage occurs to the device after this time, we are no longer responsible for the cost of repair. Repair costs can vary between €35 and €250, depending on the nature of the damage.

Repair process

If you want to have a defective device repaired, you will send the device to us. We will then repair or replace the device for you free of charge. During the repair process, you will receive a replacement device. After receiving the repaired device, you will need to return the replacement device to us.

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